Freza

Catalog

Carbide cutters
for dental laboratory

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Various colour marking helps in choosing the desired type of the cutters. A wide range of geometrical shapes and sizes provides any type of dental operations.

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Cutters with very large criss-cross toothing

Used:

for machining the super mold plaster, for working with fire-resistant masses to remove the surplus material and flow-on, and for speedy machining of plastic materials.

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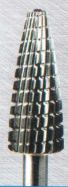
Cutters with straight-across toothing

Used:

for machining soft thermo-plastic materials of different degree of hardness, providing equal and smooth grinding surface.

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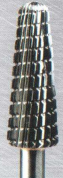
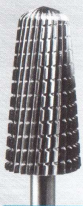
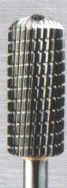
Cutters with fine spiral toothing

Used:

for machining alloys made of precious metals, metal-ceramic materials and titanium. The surfaces of the cutters are provided with fine sharp edges and the spiral system of breaking off the chips thus ensuring the economical removal of the machined material.

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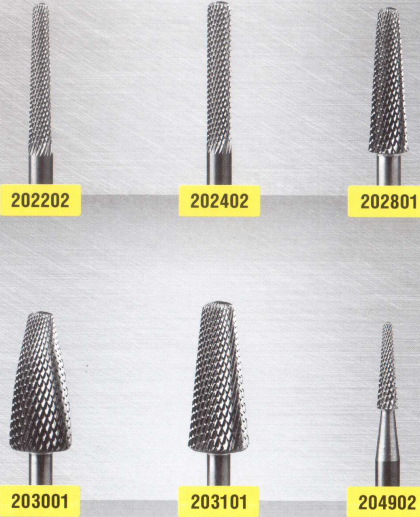
Cutters with super fine criss-cross toothing

Used:
for finishing ceramic materials, gold and alloys of precious metals.

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
Cutters with large crosswise toothing cutters with improved cutting quality

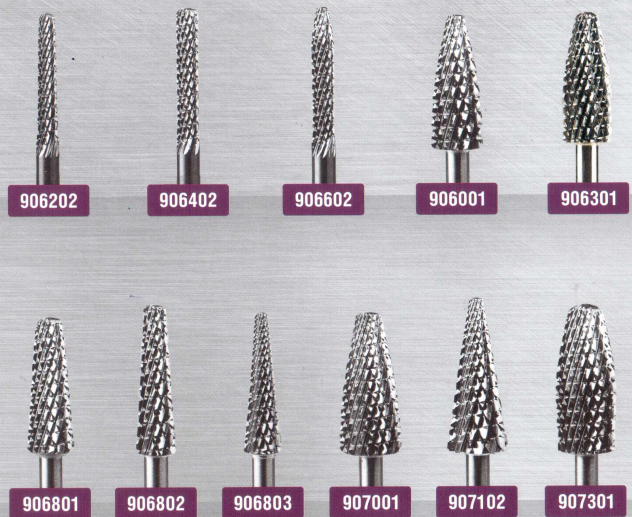Used for:
the speedy machining of chrome-cobalt, chrome-nickel and other kinds of alloys. Due to one additional cutting there are more cutting edges that make the processing easier, quicker and smoother. As a result, the quality of finished surface is improved.

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Cutters with large criss-cross toothing

Used for:

the speedy machining of chrome-cobalt, chrome-nickel and other kinds of alloys and are noted for their high wear stability. The surfaces of cutters are provided with large sharp edges and the criss-cross system of chip breaking. When they are used with high-speed dental laboratory micromotors the metal is machined very easily and rapidly.

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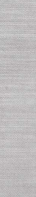
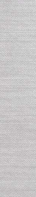
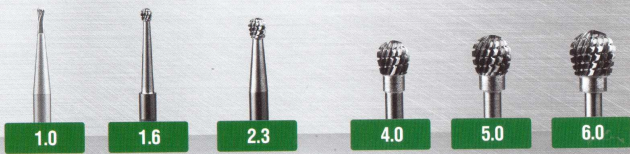
Cutters with reversive toothing

Used: for machining thermo-plastic materials of different degree of hardness. They work equally well in both directions.

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Cutters for acrylic

Used: for work with acrylic in nail service.

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Cutters for left-handers

Intended: for left-handed use.

The cutting edges are directed in the other direction, i.e. counterclock-wise.

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Cutters with fine criss-cross tothing

Used for:

finishing the surfaces of alloys made of precious metals, chrome-cobalt and chrome-nickel alloys and steel as well as for speedy rough machining of ceramic materials and imparting the shape and contouring.

Large diameter cutters can be used for machining wide surfaces and the correction of the bite. Cutters with a thin tip of the working portion are convenient for working between the elements of a denture. The cutting edges' design ensures the easy entry of the cutter into the metal in the conditions of the slight pressure onto the surface. Due to this feature these cutters may be used by doctors for the correction of solid and metal-ceramic prostheses in clinical conditions while using the micromotor of his dental unit though it is not as powerful as the dental laboratory micromotor.

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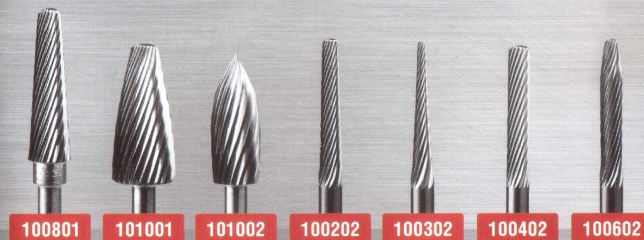
Cutters with standard fine tothing

Used for:

finishing operations and slight removal of gold, other metals and acrylic plastic materials.

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Cutters for cutting units

Used for:
high-precision modeling and machining for making special combined dentures.

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